

## Oxivir Plus J-Flex<sup>®</sup>

Broad spectrum cleaner and disinfectant

CE  
0088

### Description

Diversey **Oxivir Plus J-Flex** is a highly concentrated cleaner disinfectant for use on hard, nonporous surfaces and some soft furnishings. The J-Flex bottle delivers accurate automatic dilution.

### Key properties

- Patented Hydrogen Peroxide based acidic formulation
- Bactericidal, Virucidal, Fungicidal, Yeasticidal, and Sporicidal
- Short contact times
- Effective cleaner
- Fragrance free

Packaging	Flow rate	Amount of RTU solution (Litres per pack)
1.5L	Low	57 triggers of 750ml
1.5L	High	43 L of solution

### Benefits

- Cleans and disinfects in one step, reducing working time
- Extensively tested against European norms and effective against multiple pathogens of concern, including HIV, HBV, HCV, Avian Influenza, Pseudomonas aeruginosa, Staphylococcus aureus, Salmonella enterica, MRSA, VRE and Norovirus
- Excellent cleaning performance, including descaling and lime limitation
- Good user safety profile, with no hazard warnings on RTU format; does not contain respiratory irritants
- Good environmental profile. Active breaks down into oxygen and water
- Suitable for use in high risk areas in Healthcare, including OR, ICU, NICU and patient rooms

### Use instructions

**Dosage:** Rotate selector to either low or high dilution and fill spray bottle or bucket with J-Flex bottle. Both settings give the same dilution to guarantee disinfection at the proper dosage.

#### One Step Application:

##### Spray cleaning and disinfection:


1. Remove any gross soil
2. Spray onto surfaces
3. Wipe with clean cloth

##### Bucket cleaning and disinfection:

1. Remove any gross soil
2. Apply solution with mop/cloth and allow to air dry
3. Leave surface wet for desired application time
4. Prepare fresh solution if solution becomes visibly dirty

**Note:** To disinfect for Norovirus, pre clean the surface prior to disinfection. Allow surface to remain wet for two minutes.

**Important:** Do not use warm water. Do not mix with other products. Do not use on surfaces sensitive to acid.

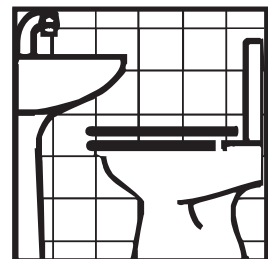

# Oxivir Plus J-Flex<sup>®</sup>

Broad spectrum cleaner and disinfectant

## Technical Data

Appearance:	Clear yellow liquid
Relative density [20°C]:	1.029
pH-value in use:	1.8 +/- 0.5 at 3.5% dilution
pH-value neat:	0.5

The above data is typical of normal production and should not be taken as a specification.

Product Code	Pack Size	Cases per layer	Cases per pallet	Selling bar code
100829921	1 x 1.5L	45	180	7615400163533

## Safe handling and storage information

Full guidance on the handling and disposal of this product is provided in a separate Material Safety Data Sheet. Store in original container. Avoid extremes of temperature. Reacts with oxydisable substances.

## Product compatibility

Under recommended conditions of use, Diversey **Oxivir Plus J-Flex** is suitable for use on most materials commonly encountered.

## Environmental information

The surfactants used in this product are ultimately biodegradable in compliance with the requirement of the EU Detergent Regulation, EC 648/2004.

## Microbiological data

Efficacy proven by compliance with: EN1276, EN13697, EN1650, EN14776, EN13624 and EN13727.